



## **Lifeline® "Oil-Less" Claw Medical Vacuum Base Mount Single Point Connection (SPC) Triplex System (5 Hp – 15 Hp)**

### **SPECIFICATION**

#### **Single Point Connection (SPC) System Design**

The Lifeline® Oil-Less Claw Medical vacuum package is fully compliant with NFPA 99 and features a common base with single point connections for the electrical panel, intake, and discharge. Each pump and the receiver are connected to a common intake manifold. Each pump is connected to a common discharge manifold. The common base is separable for transport through standard 34.5" doorway. Designed and manufactured with ISO 13485 processes, each system is completely tested before shipment and includes:

- Three "oil-less" claw rotary vacuum pumps with three motors
- Integral pre-wired control panel
- Vertical air receiver with full-size three-valve bypass system sized for appropriate demand
- OSP-322-10 seismic pre-certification rating of 2.5 S<sub>DS</sub>

#### **Vacuum Pump**

Each pump is a direct driven, non-contacting claw type, capable of operating continuous duty at the following levels:

- 5.4 Hp at 28.4" Hg (sea level)
- 6.4 and 7.5 Hp at 27" Hg (sea level)
- 8.7 Hp at 25.5" Hg (sea level)
- 10 Hp and 15 Hp at 24" Hg (sea level)

The pumping chamber is oil free. The pump is completely aircooled with no water requirements. Each pump contains:

- Vacuum relief valve
- Check valve to prevent backflow through off-cycle units
- Flexible connector and isolation valve
- High discharge temperature sensor
- Oil drain valve and oil sight glass

#### **Vacuum Pump Motor**

Motors are continuous duty, C-face, TEFC, suitable for 230/460 or 208V, 60 hertz, 3-phase electrical service and 380V, 50 hertz, 3-phase electrical service.

#### **Vacuum Filtration Per NFPA 99**

A HEPA inlet air filter, 0.3 micron, 99.97% efficiency, is mounted before each vacuum pump. A clear, glass collection canister is mounted below each HEPA filter, with quarter turn valve to isolate canister from filter during service. The inlet filter canister contains a bleed valve to relieve vacuum before servicing.

#### **Intake Piping**

Each vacuum pump has a factory piped intake with integral flex connector, isolation valve, and check valve. Interconnecting piping consists of powder-coated steel tubing and flanges.

#### **Vacuum Receiver**

The vacuum receiver is ASME Code stamped, and rated for a minimum 150 PSIG design pressure. The receiver has a full-size three-valve bypass system to allow for draining of the receiver without interrupting the vacuum service. A manual drain is provided on the receiver.

#### **Exhaust Piping**

Each vacuum pump is factory piped to an exhaust manifold with integral flex connector and drip leg with ball valve and condensate drain. Interconnecting piping consists of powdercoated steel tubing and flanges. An exhaust muffler is shipped loose.

#### **TotalAlert 360 Control System**

The triplex mounted and wired TotalAlert 360 control system is U.L. labeled. The control system provides automatic lead/lag sequencing and automatic alternation of all vacuum pumps based on first-on/first-off principle with provision for simultaneous operation if required. Automatic activation of reserve unit, if required, will activate an audible alarm as well as a visual alarm on the display screen.

#### **Additional components include:**

- NEMA 12 control panel enclosure
  - Circuit breaker disconnects for each motor with external operators
  - Full voltage motor starters with overload protection
  - 24V control circuit
  - 65kAIC SCCR rating for control cabinet
- The touch screen controls feature a 10" color, high resolution screen. Screen displays and functions include:
- Easy to read system vacuum level
  - Status of all units (Running, Available, Off, next to Run)
  - Trend graphs for vacuum level and units running
  - Run time hour meters for each unit
  - Visual/audible alarm indications with isolated contacts for all standard remote alarms
  - Event log recording alarms and system activity
  - Service alerts
  - Event log recording service warnings and service history
  - Integral cellular connectivity to MyMedGas, allowing electronic notifications of alarms and warnings
  - Daily rounds uploadable to MyMedGas
  - BTL listed, BACnet/IP communication


#### **Installation**

The installation of this vacuum technology is limited to 4,000 feet elevation above sea level. For installation of this equipment above this elevation, please contact the factory.




## Standard Configuration

15 Hp Triplex

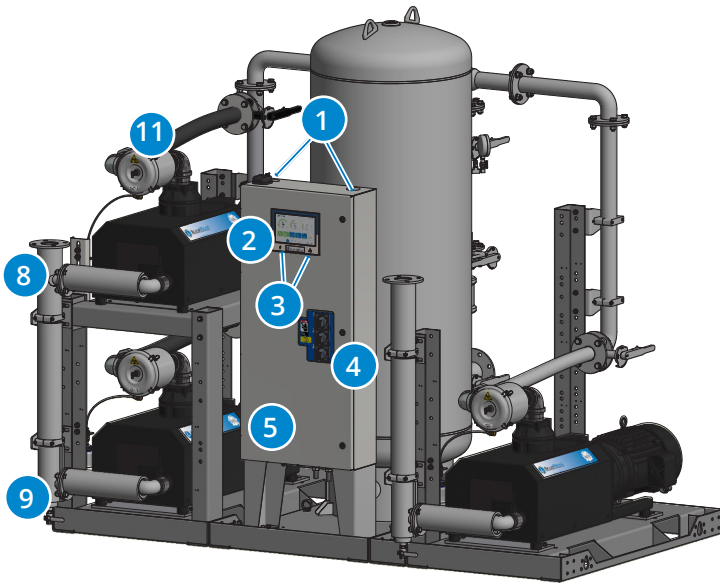

- 8** Discharge manifold with a single point connection to the exhaust piping, with flex hoses factory installed.
- 9** Factory installed exhaust drip leg with ball valve and condensate drain.
- 10** Flex connectors on vacuum inlet and discharge to isolate pump vibration from facility pipeline.
- Individual HEPA air inlet filter per vacuum pump to prevent contaminants from entering vacuum pump, ensuring safety during servicing. Housing bleed valve to remove vacuum for filter replacement.
- 11** Glass collection canister mounted below HEPA filter, with isolation valve to allow for easy changeout.
- Easy to disassemble for transport through doorways and reassemble as Single Point Connection for installation.
- 12**
- Openings in base frame to allow access underneath pumps for pump maintenance.
- 13**
- 14**

- 1** Control panel pre-drilled for power and alarms for easy electrical installations. Ethernet connection on panel top for easy installation of BACnet connectivity, with additional connection on the side panel for software updates.
- 2** TotalAlert 360 controls featuring 10" color screen with high resolution 1280 x 800 Capacitive Touch Display for exceptional clarity, visibility, and useability.
- 3** LED indication of Reserve in Use condition as backup annunciation to display failure and LED indication of power to the control panel.
- 4** Through the door disconnects for pumps add to safety during service.
- 5** Control cabinet components achieve rating of 65kAIC SCCR.
- 6** Vacuum switch as backup to vacuum transducer. In case of transducer failure, system operates effectively.
- 7** Full-size 3-valve receiver bypass with flanged valves reduce potential leak points over NPT fittings.

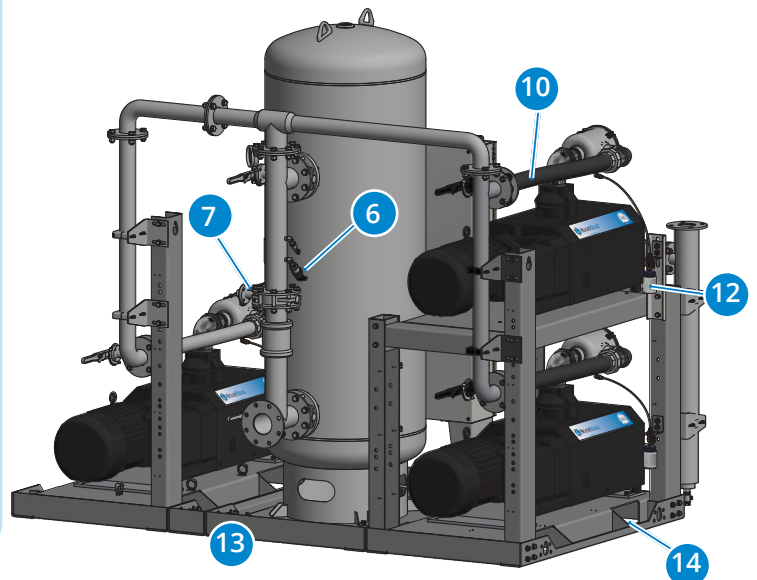

## **TotalAlert 360 Control System**



## **TotalAlert 360 10" Touch Screen**

- 10" LCD 1280 x 800 Capacitive Touch Display for exceptional clarity and visibility, with enhanced user experience
- LCD is IPS-TFT for enhanced color and contrast, featuring 85% viewing angle in all directions
- Toolbars on all screens with easy access navigation icons that enable full access with minimal touches

## **MyMedGas Connectivity**

- Cellular connectivity to cloud-based MyMedGas web page for remote operator to view system controls and display information
  - » View remotely system operation details, alarms and event history
  - » View remotely maintenance timers, service history logs
  - » Easily download system trends, event logs and reports for system management and record keeping
- Electronic notification
  - » Allows for remote alerts of alarm and warning conditions
  - » Allows for remote alerts of routine maintenance
- Logging rounds to MyMedGas
  - » Main screen touchpoint for MyMedGas with time stamp to log operating conditions, system parameters during daily rounds
  - » Automated report generated to MyMedGas

## **Ethernet Connectivity**

- Control panel contains Ethernet port on top of cabinet for easy installation of BACnet
- TCP/IP protocol for BACnet activation
- Ethernet port on side of control cabinet that allows reprogramming with a standard computer for software updates

## **Control Cabinet Safety**

- Volt free relay contacts for all standard alarms
- Low voltage (24V) control circuit
- Full voltage motor starters with overload protection
- Circuit breaker disconnects for each compressor
- 65kAIC SCCR rating

## **Redundancy**

- Each vacuum pump has an individual board for control allowing units to function independently
- In unlikely malfunction of display board or display screen, system will function normally and activate alarm
- If master board malfunctions, system goes to failsafe operation with backup vacuum switch
- Independent LED indicators on control screen for Reserve in Use and power to the control panel as backup annunciation to display failure

## **Master Display Screen**

- Main Screen
  - » Displays the system operating conditions, including system vacuum level
  - » Displays the pump unit sequence, including status of pumps (running, available, off) and next to start
- Trends & Graphs
  - » Shows measured values of the system operating conditions over a period of time (vacuum level, ambient temperature)
  - » Shows unit usage and operation, clearly identifying each unit and its running status
  - » Selectable time periods consist of 60 minutes, 4 hours, 24 hours, and 6 days
  - » Visually precise with green and red shading for normal and non-normal areas
- Service
  - » Displays ambient temperature in the room
  - » Maintenance screen shows suggested and required maintenance items with resettable timers
  - » Historical event log records all service activities
  - » Link to operations manuals associated with system
- Alarms and Shutdowns
  - » All system/unit Alarms and Shutdowns displayed with visual indication (Green or Red) and unit operation status, allowing for quick identification and evaluation of system operation
  - » Custom messaging for each alarm and shutdown
  - » Testing mode enables operator to test all alarm/shutdown events (password protected), with all test events recorded in the event log
  - » Event History Log records all system event history
- Unit Operation
  - » Screens show the operation mode of the unit along with automatic or manual mode setting
  - » Unit status screen displays the running hour meter values, including average daily run time and average starts/hour
  - » Pump rotation allows the unit to run for a short period to check pump rotation
- System Settings
  - » Allows the adjustment of system pressure operating range
  - » Displays pertinent system model information

**SPECIFICATION TABLE**


Vacuum System Specifications <sup>1</sup>										
System Model No.	HP	Capacity <sup>2</sup> @19" Hg (scfm)		System <sup>3</sup> BTU/Hr	Receiver (gallons)	Noise <sup>4</sup> Level	System FLA			System Weight (lbs)
		Pump	System				208V	230V	460V	
SPC Triplex - 60 Hz										
VHS05T-120V-T	5.4	38	76	21,989	120	82	43.2	39.7	20.1	2,520
VHS05T-200V-T	5.4	38	76	21,989	200	82	43.2	39.7	20.1	2,745
VHS05T-240V-T	5.4	38	76	21,989	240	82	43.2	39.7	20.1	2,821
VHS06T-200V-T	6.4	52	104	26,061	200	82	54.3	53.5	27.0	3,454
VHS06T-240V-T	6.4	52	104	26,061	240	82	54.3	53.5	27.0	3,529
VHS07T-200V-T	7.5	65	130	30,133	200	82	74.7	60.7	30.6	3,466
VHS07T-240V-T	7.5	65	130	30,133	240	82	74.7	60.7	30.6	3,541
VHS08T-200V-T	8.7	77	154	35,426	200	85	77.4	73.9	37.2	3,495
VHS08T-240V-T	8.7	77	154	35,426	240	85	77.4	73.9	37.2	3,570
VHS10T-200V-T	10	87	174	40,720	200	90	83.7	72.7	36.6	4,366
VHS10T-240V-T	10	87	174	40,720	240	90	83.7	72.7	36.6	4,434
VHS15T-200V-T	15	128	256	61,080	200	90	123.6	106.9	53.7	4,530
VHS15T-240V-T	15	128	256	61,080	240	90	123.6	106.9	53.7	4,591
SPC Triplex - 50 Hz							380V			
VHS05T-120V-TE	5.4	32	64	21,989	120	82	18.3			2,520
VHS05T-200V-TE	5.4	32	64	21,989	200	82	18.3			2,745
VHS05T-240V-TE	5.4	32	64	21,989	240	82	18.3			2,821
VHS06T-200V-TE	6.4	43	86	26,061	200	82	29.7			3,454
VHS06T-240V-TE	6.4	43	86	26,061	240	82	29.7			3,529
VHS07T-200V-TE	7.5	54	108	30,133	200	82	32.1			3,466
VHS07T-240V-TE	7.5	54	108	30,133	240	82	32.1			3,541
VHS08T-200V-TE	8.7	64	128	35,426	200	85	39.9			3,495
VHS08T-240V-TE	8.7	64	128	35,426	240	85	39.9			3,570
VHS10T-200V-TE	10	72	144	40,720	200	90	43.5			4,366
VHS10T-240V-TE	10	72	144	40,720	240	90	43.5			4,434
VHS15T-200V-TE	15	103	206	61,080	200	90	45.0			4,530
VHS15T-240V-TE	15	103	206	61,080	240	90	45.0			4,591

**Specification Table Notes:**

1. Normal operating conditions at a maximum ambient of 105° F. Consult factory for higher ambient conditions.
2. All capacities are shown as NFPA 99 system capacities (reserve vacuum pump on standby).
3. All system BTU/HR are shown with reserve vacuum pump on standby.
4. All noise levels are shown in dB(A) and reflect reserve pump on standby.

### Standard Configuration

8.7 Hp Triplex

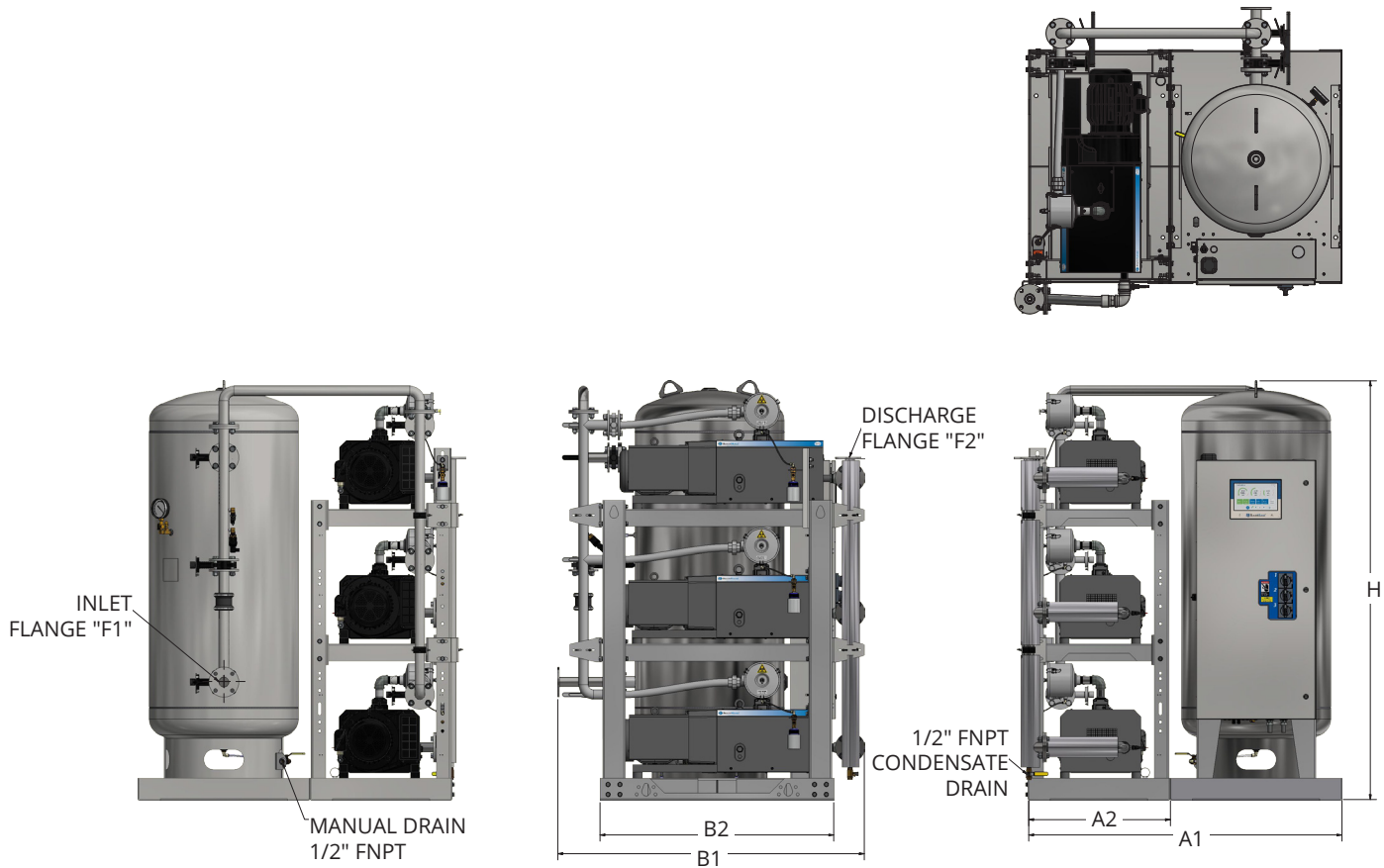


System Model No.	HP	Dimensions						
		A1 (Length)	A2 (Length)	B1 (Width)	B2 (Width)	H (Height)	F1 (Inlet Flange)	F2 (Discharge Flange)
VHS05T-120V-T	5.4	63	28.5	61.64	47	83.12	2	2
VHS05T-200V-T	5.4	63	28.5	61.64	47	84.22	2	2
VHS05T-240V-T	5.4	63	28.5	61.64	47	96.21	2	2
VHS06T-200V-T	6.4	66	31.5	67.18	52.5	84.35	3	3
VHS06T-240V-T	6.4	66	31.5	67.18	52.5	96.34	3	3
VHS07T-200V-T	7.5	66	31.5	67.18	52.5	84.35	3	3
VHS07T-240V-T	7.5	66	31.5	67.18	52.5	96.34	3	3
VHS08T-200V-T	8.7	66	31.5	67.18	52.5	84.35	3	3
VHS08T-240V-T	8.7	66	31.5	67.18	52.5	96.34	3	3

#### Notes:

- Discharge muffler ships loose
- Allow 36 inches in front of control panel for maintenance and ventilation, all other sides require 24 inches of clearance.
- Additional drawings/diagrams available for download at [www.beaconmedaes.com](http://www.beaconmedaes.com).

### Standard Configuration

15 Hp Triplex

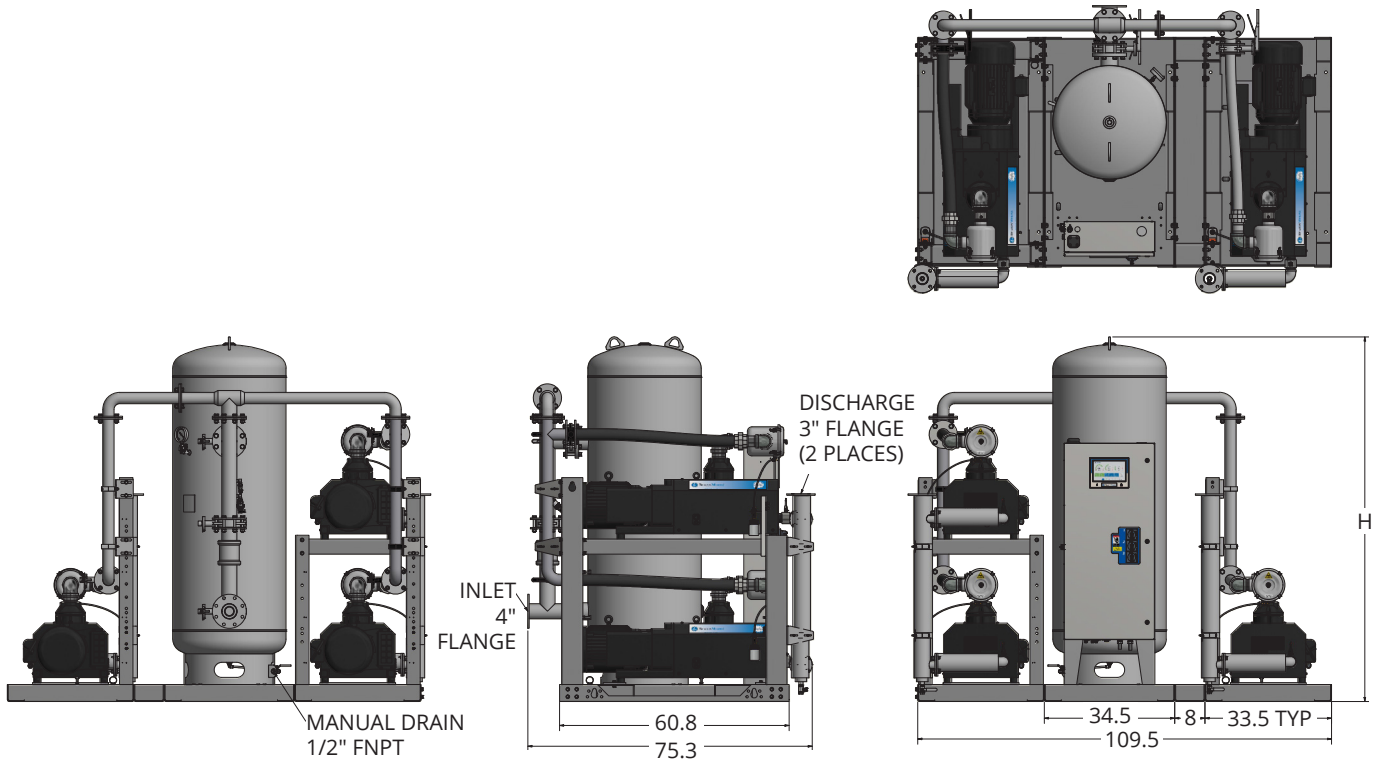

System Model No.	HP	Dimensions
		H (Height)
VHS10T-200V-T	10	84.48
VHS10T-240V-T	10	96.48
VHS15T-200V-T	15	84.48
VHS15T-240V-T	15	96.48

#### Notes:

- Discharge muffler ships loose
- Allow 36 inches in front of control panel for maintenance and ventilation, all other sides require 24 inches of clearance.
- Additional drawings/diagrams available for download at [www.beaconmedaes.com](http://www.beaconmedaes.com).

*Life is in the details.®*