

Emergency Reserve Manifold

Description

The Pneumatech MGS HTM 02-01 Emergency Reserve Manifold (ERM) is a safe means of providing a continuous supply of medical gas from high pressure cylinders to a pipeline system at the required distribution pressure of 400 kPa, 700kPa or 1,100 kPa. The HTM 02-01 ERM comprises two separate stages of pressure regulation enabling high peak flow rates without significant pressure reduction downstream. The controlling regulator assemblies are mounted on a control panel, which alternately directs gas from two banks of high pressure cylinders (duty and standby). Changeover is achieved manually when one bank of cylinders is empty.

The HTM 02-01 ERM is typically used as a reserve or backup gas supply, usually to a high demand system i.e. an automatic changeover manifold, intended to supply oxygen, nitrous oxide, O2/N2O (50%/50%), medical air, surgical air, carbon dioxide and nitrogen installations complying with UK Department of Health (DoH) Health Technical Memorandum (HTM)No. 02-01, BS EN ISO 7396-1 or similar guidelines or standards.

The HTM 02-01 ERM consists of two main components:

- The control/regulator assembly regulates and controls the flow of gas from the cylinder header manifold.
- Integrated cylinder header manifolds support high pressure cylinders in racks and provide a gas specific connection, by means of manifold headers and tailpipes to either side of the control/regulator assembly.

Classification

- Emergency Reserve Manifolds are designed to conform with the following standards: HTM 02-01, BS EN ISO 7396-1, BS EN ISO 15001 and BS EN ISO 10524-2
- CE marked under the Medical Device Directive 93/42/EEC with approval from notified body no. 0088 (Lloyd's Register Quality Assurance)
- · Class IIb Medical device

Construction

- Mild steel powder coated back plate
- All components degreased for oxygen use

Features

- Central regulator panel with 2x1 cylinder headers each side as standard
- Modular header manifolds
- Headers are complete with gas specific cylinder connections for flexible cupronickel tailpipes
- Non-return valves are fitted to each tailpipe connection
- Pin indexed tailpipes in compliance with BS EN ISO 407
- Duty bank manually selected by opening the isolation diaphragm valve on the central panel
- Pre-wired terminal block for alarm output to the automatic changeover manifold
- · East terminal unit test point fitted
- Integrated non-return valve and lockable line isolation valve for connection to the distribution system

Technical Datasheet



Services for Use

- Oxygen
- Nitrous Oxide
- O2/N2O (50%/50%)
- Medical & Surgical Air
- Carbon Dioxide
- Nitrogen

Mounting

The complete manifold shall be fitted to a wall mounting plate attached to the wall with four screws.

Pressure Gauges and Switches

ERM: Two pressure gauges with contact switches installed on each isolation valves monitor both banks of cylinders factory set to 68 bar (14 bar for Nitrous Oxide) to allow remote indication of 50% plus capacity: 'Reserve Low' or 'Reserve Fault' alarm conditions. **ERM Lite**: Two pressure gauges installed on each isolation valves. One pressure gauge with contact switch installed to monitor duty bank of cylinders and factory set to 68 bar (14 bar for Nitrous Oxide) to allow remote indication of 50% plus capacity: 'Reserve Low' or 'Reserve Fault' alarm conditions.

Pressure Regulation

- Maximum inlet pressure: 23,000 kPa (230 bar)
- Outlet pressure reduced to: 400 kPa, 700 kPa or 1,100 kPa

Flow Rate

The manifold control system shall be capable of supplying a flow of:

- 1,200 l/min to a nominal 400 kPa distribution system
- 2,000 l/min to a nominal 700 kPa distribution system
- 2,000 l/min to a nominal 1,100 kPa distribution system

(Note! Based on a 10% reduction in flowing pressure from a static pressure set point).

For small installations (required flow less than 200 l/min) it is allowed to use, simplified version of Emergency Reserve Manifold Lite with multistage regulator (ERM Lite).

Header Rack Services

- Gas specific thread types:
- Nitrous Oxide
 Nitrous Oxide
 Oxygen
 Air
 O2/N2O
 CO2
 Nitrogen
 M18 x 2
 M20 x 2
 M24 x 2
 O2/N2O
 M22 x 2
 M3/8 inch BSP
 M14 x 2



















- Email: sales@p-mgs.com www.p-mgs.com
- \bullet The right is reserved to change this technical specification without prior notice



Technical Datasheet

Emergency Reserve Manifold 2x1 Installation with J or G Cylinders and VF Cylinders

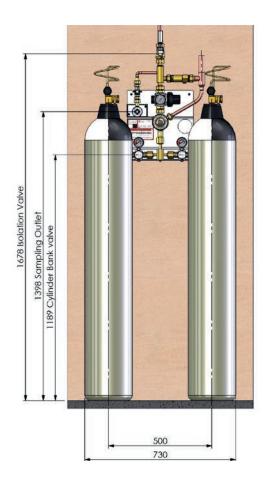

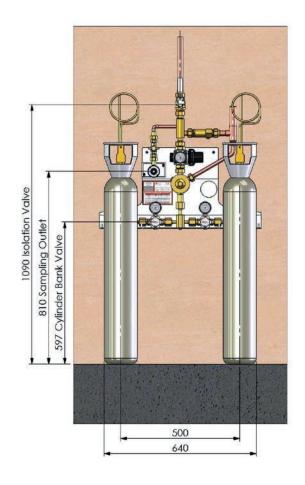


Weight & Dimensions

Emergency Reserve Manifold Installation with 2x2 extension



Manifold Size (No. of Cylinders)	Total Width (mm)		
2 x 1	730		
2 x 2	1200		
2 x 3	1942		
2 x 4	2269		
2 x 5	2952		
2 x 6	3279		

Note: Dimensions for ERM and ERM Lite versions are identical.

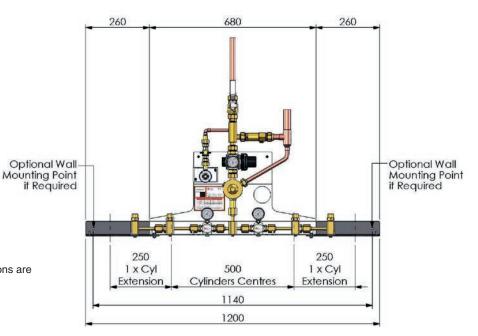



- Email: sales@p-mgs.com www.p-mgs.com
- The right is reserved to change this technical specification without prior notice


















Technical Datasheet


Emergency Reserve Manifold Installation with 2x3 extension

371 260 680 260 371 260 250 250 250 1 x Cyl 1 x Cyl Extension Extension Extension Extension Extension Extension


Emergency Reserve Manifold Installation with 2x4 extension

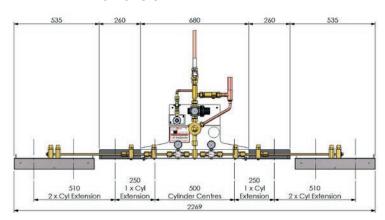

Emergency Reserve Manifold Installation with 2x5 extension

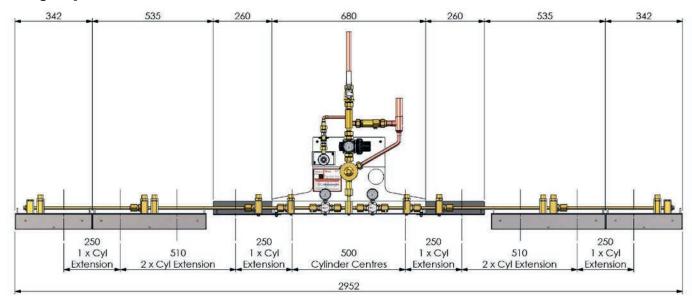

Emergency Reserve Manifold Installation with 2x6 extension

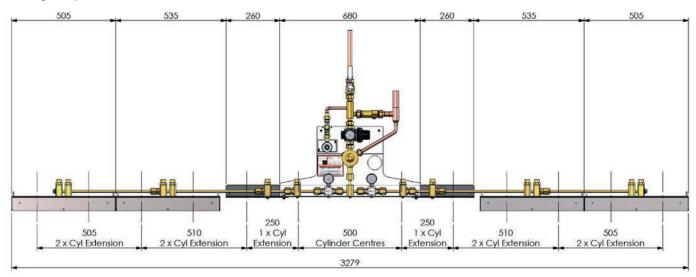



- Email: sales@p-mgs.com www.p-mgs.com
- The right is reserved to change this technical specification without prior notice

















Technical Datasheet

Relief Valve Settings

Nominal 400 kPa ERM
 Nominal 700 kPa ERM
 Nominal 1,100 kPa ERM
 1,300 kPa
 1,300 kPa

Ordering

Gas ID	Control Panel 2x1		Extension Kit	1 Cyl. Extension	2 Cyl. Extension	
	ERM	ERM Lite	2x1, Part No	- one side, Part No	- one side Part No	
Oxygen	6000530HP	8102341727	2005826	2000232	2000204	
Nitrous Oxide	6000531HP	8102341728	2005827	2000233	2000205	
Oxygen / Nitrous Oxide	6000532HP	8102341729 2005828		2000234	2000206	
Medical Air	6000533HP	8102341730				
Surgical Air, 7 bar	6000534HP	8102341731	2005829	2000235	2000207	
Surgical Air, 11 bar	6000535HP	n/a				
Nitrogen, 7 bar	6000536HP	8102341733	2005830	2005110	2005108	
Nitrogen, 11 bar	6000537HP	n/a	2003830	2003110		
Carbon Dioxide	6000538HP	8102341732	2005831	2000242	2000243	

Tailpipes


Tailpipe	O ₂	N ₂ O	N ₂ O / O ₂	Air	CO ₂	N_2
Pin-Indexed (ISO 407) ²	8102340110	8102340123	8102340130	8102340140	8102340151	
Pin-Indexed (ISO 407) Extended	8102340116		8102340131	8102340146		
Bull nose (ISO5145) 2,3, Side entry	8102369663		8102369664			
Bull nose (BS341) ¹ Top entry	8102340111			8102340141		8102340161
Bull nose (BS341) ¹ Side entry	8102340112	8102340120		8102340142	8102340150	
Bull nose (BS341) ¹ Extended	8102340117	8102340125		8102340147	8102340154	
US Std (CGA)	8102340114	8102340122		8102340144		
Chinese Bullnose	8102340115	8102340124		8102340145	8102340152	8102340162

Notes:

- 1. Bullnose tailpipes (except Chinese type) are to the following BS standards: Oxygen, Air, Nitrogen: BS: 341-1 No. 3; Carbon dioxide: BS: 341-1 No. 8; Nitrous oxide: BS: 341-1 No. 13.
- 2. Mixture N2O O2, (registered trade name Entonox BOC) low pressure cylinder "G" type has Pin-indexed connector according to standard BS EN ISO 407 and 230 bar cylinder "EW" type has Bull nose connector according to ISO 5145 No. 13.
- 3. Oxygen cylinder "J" type has Pin-Indexed connector according to ISO 407, where "W" type (230 bar) has Bull nose according to ISO 5145 No. 5.

Accessories

Description	Part No	
Manifold Header Corner Connector - one side	2000227	

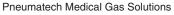

- Email: sales@p-mgs.com www.p-mgs.com
- The right is reserved to change this technical specification without prior notice

