

NOTES:

- 1. ALL INTERCONNECTING PIPE AND FITTINGS SHALL BE INSTALLED BY THE CONTRACTOR.
- 2. ALL PIPING SHALL BE INSTALLED IN ACCORDANCE WITH NFPA 99. PIPING LAYOUT IS DIAGRAMMATIC ONLY. FINAL ARRANGEMENT WILL BE SITE SPECIFIC. CONTRACTOR WILL BE RESPONSIBLE FOR FIELD VERIFICATION AND COORDINATING ACTUAL LOCATION WITH ALL OTHER TRADES.
- 3. CONTRACTOR SHALL VERIFY MOTOR VOLTAGE, PHASE AND AMP RATINGS BEFORE STARTING ELECTRICAL INSTALLATION; AND MAKE CERTAIN THE VOLTAGE SUPPLIED BY THE HOSPITAL IS THE SAME.
- 4. THE ROOM REQUIRES DEDICATED MECHANICAL VENTILATION WITH AN ALLOWABLE TEMPERATURE VARIANCE OF 40° TO 105°F. (FOR HIGHER AMBIENTS, CONSULT FACTORY)
- 5. ELECTRICAL POWER SHALL BE SUPPLIED FROM THE EQUIPMENT SYSTEM BRANCH OF THE ESSENTIAL ELECTRICAL SYSTEM (EMERGENCY POWER).
- 6. EQUIPMENT, INSTALLATION, AND TESTING OF THE MEDICAL VACUUM SYSTEM SHALL COMPLY WITH NFPA 99 AND ALL STATE AND LOCAL CODES OR ORDINANCES.
- 7. WIRE LAG PUMP RUNNING ALARM REMOTE CONTACTS IN THIS CONTROL SYSTEM TO MASTER ALARM SYSTEM AS REQUIRED BY NFPA 99.
- 8. ALL ACCESSORIES, PIPE AND FITTINGS, EXCEPT DISCHARGE SILENCERS, BEYOND INLET AND DISCHARGE PORTS ARE SUPPLIED AND INSTALLED BY OTHERS.
- 9. WHEN DETERMINING THE TOTAL PIPE LENGTH, ADD ALL THE STRAIGHT LENGTHS OF PIPE TOGETHER IN ADDITION TO THE NUMBER OF ELBOWS TIMES THE THE EFFECTIVE PIPE LENGTH FOR THAT PIPE SIZE. (SEE EQUIVALENT PIPE LENGTH TABLE & EXAMPLE CALCULATION IN THE O&M MANUAL)
- 10. ALWAYS CONSIDER SURROUNDING OCCUPANCIES FOR ALL ROTATING EQUIPMENT. ADDITIONAL ISOLATION/VIBRATION TRANSLATION REQUIREMENTS DUE TO SYSTEM LOCATION SHOULD BE DETERMINED AND PROVIDED BY OTHERS.

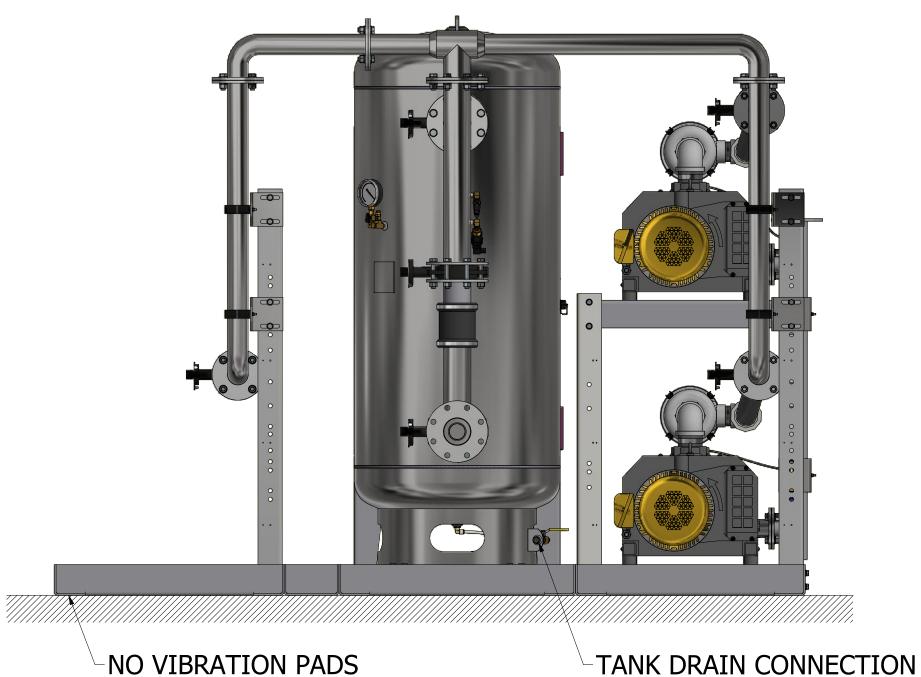

ARE NECESSARY AT THIS LOCATION

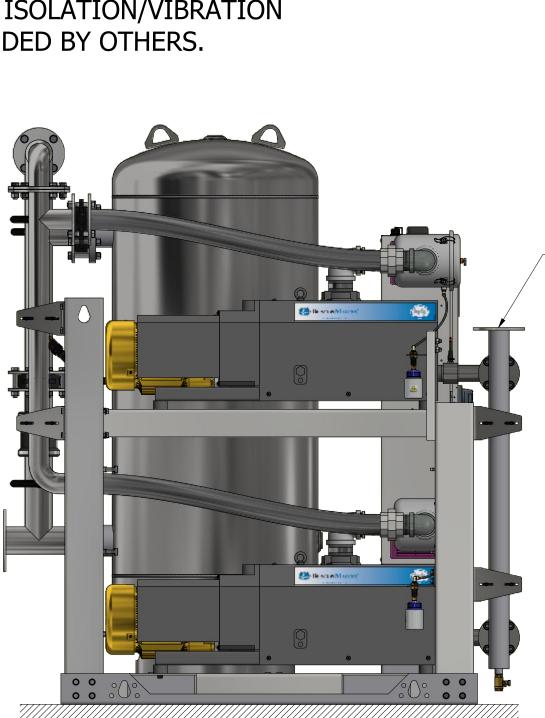

FOR INTAKE PIPE SIZING REFER TO STANDARD-PRESSURE DROP TABLES. TOTAL PRESSURE DROP SHOULD BE LESS THAN 2" Hg AT A SYSTEM VACUUM LEVEL OF 15" Hg.

EXHAUST PIPE SIZE TABLE

11.9

FROM HOSPITAL

EFFECTIVE PIPE LENGTH FOR ELBOWS PIPE SIZE (IN) 3.00" NPT 4.00" NPT 5.00" NPT

PIPE LENGTH

0-50' 51'-200' 201'-500'

(SEE NOTE 9)

TX 10 | 3.00" NPT | 4.00" NPT | 5.00" NPT

7.9

UNIT SIZE

EFF. PIPE

LENGTH (FT)

FIELD PIPING INLET FIELD PIPING OUTLET

10

FOR FUTURE **EXPANSION PUMP**

-MAIN LINE VACUUM SWITCH

-MAIN VACUUM GAUGE

-SOURCE SHUTOFF VALVE

(4" FLANGE)

HOSPITAL CONNECTION

-ISOLATION VALVE

PUMP SINGLE POINT POWER WIRING ACCESS

-FUTURE EXPANSION

OUTSIDE, TURN DOWN AND SCREEN PER NFPA 99

PIPE EXHAUST

ETHERNET WIRING ACCESS-

ALARM WIRING ACCESS

-3" DISCHARGE FLANGE

3" EXHAUST SILENCER



-DRIP LEG VALVE

This drawing and the information contained thereon remain the property of BeaconMedaes and may not be used for other than the purpose for which it is loaned without the expressed written permission **INSTALLATION DIAGRAM 10HP** DX-EXP-TX CLAW VAC VSD SPC STANDARD AND O2 ASSURED

(200 GALLON RECEIVER SHOWN)

Form F-007 Rev. 01