


his drawing and the information contained thereon remain the property of eaconMedæs and may not be used for other than the purpose for which it loaned without the expressed written permission from BeaconMedæs Engine MAIN VACUUM GAUGE~ **NOTES:** SOURCE SHUTOFF VALVE -1. ALL INTERCONNECTING PIPE AND FITTINGS SHALL BE INSTALLED BY THE CONTRACTOR. INLET-2. ALL PIPING SHALL BE INSTALLED IN ACCORDANCE WITH NFPA 99. PIPING LAYOUT IS DIAGRAMMATIC ONLY. 2" FLANGE

- FINAL ARRANGEMENT WILL BE SITE SPECIFIC. CONTRACTOR WILL BE RESPONSIBLE FOR FIELD VERIFICATION AND COORDINATING ACTUAL LOCATION WITH ALL OTHER TRADES.
- 3. CONTRACTOR SHALL VERIFY MOTOR VOLTAGE, PHASE AND AMP RATINGS BEFORE STARTING ELECTRICAL INSTALLATION; AND MAKE CERTAIN THE VOLTAGE SUPPLIED BY THE HOSPITAL IS THE SAME.
- 4. THE ROOM REQUIRES DEDICATED MECHANICAL VENTILATION WITH AN ALLOWABLE TEMPERATURE VARIANCE OF 40° TO 105°F. (FOR HIGHER AMBIENTS, CONSULT FACTORY)
- 5. ELECTRICAL POWER SHOULD BE SUPPLIED FROM THE EQUIPMENT SYSTEM BRANCH OF THE ESSENTIAL ELECTRICAL SYSTEM (EMERGENCY POWER).
- 6. EQUIPMENT, INSTALLATION, AND TESTING OF THE MEDICAL VACUUM SYSTEM SHALL COMPLY WITH NFPA 99 AND ALL STATE AND LOCAL CODES OR ORDINANCES.
- 7. WIRE LAG PUMP RUNNING ALARM REMOTE CONTACTS IN THIS CONTROL SYSTEM TO MASTER ALARM SYSTEM AS REQUIRED BY NFPA 99.
- 8. ALL ACCESSORIES, PIPE AND FITTINGS BEYOND INLET AND DISCHARGE PORTS ARE SUPPLIED AND INSTALLED BY OTHERS.
- 9. WHEN DETERMINING THE TOTAL PIPE LENGTH, ADD ALL THE STRAIGHT LENGTHS OF PIPE TOGETHER IN ADDITION TO THE NUMBER OF ELBOWS TIME THE EFFECTIVE PIPE LENGTH FOR THAT PIPE SIZE. (SEE EQUIVALENT PIPE LENGTH TABLE & EXAMPLE CALCULATION IN THE O&M MANUAL)

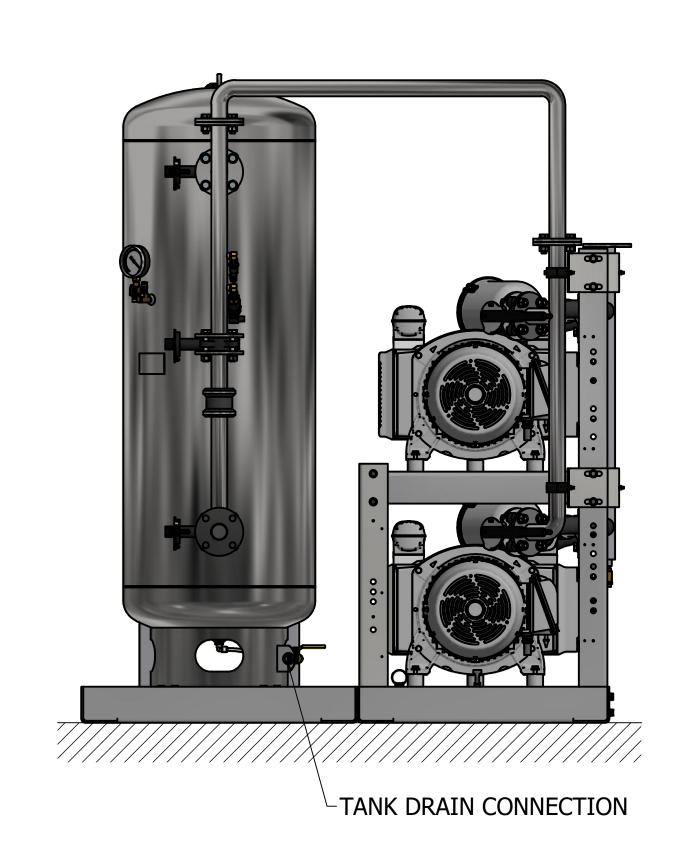

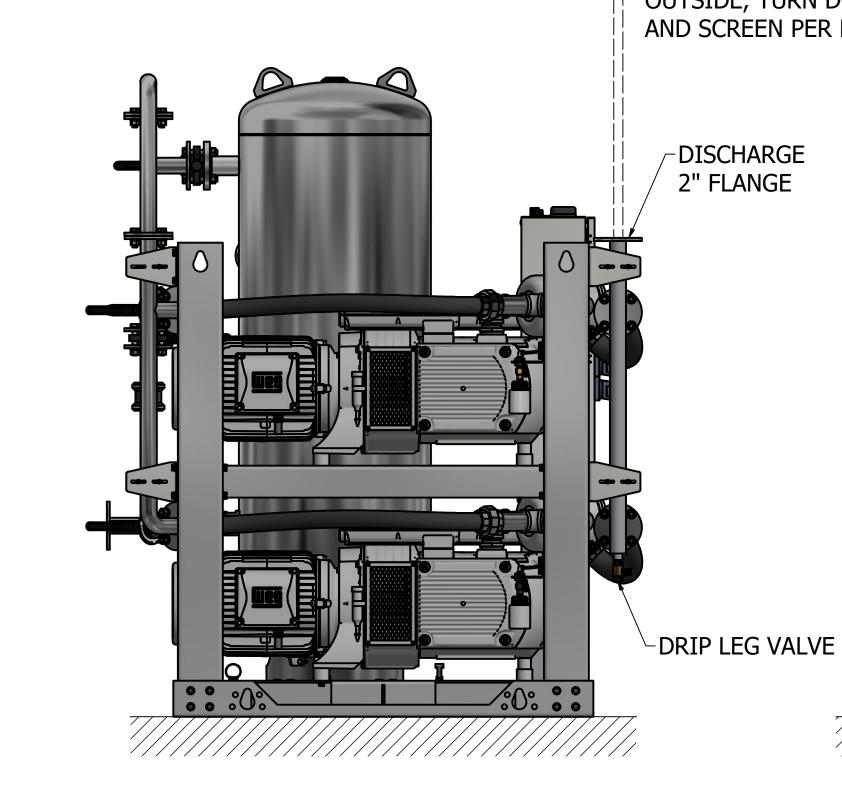

PIPE EXHAUST

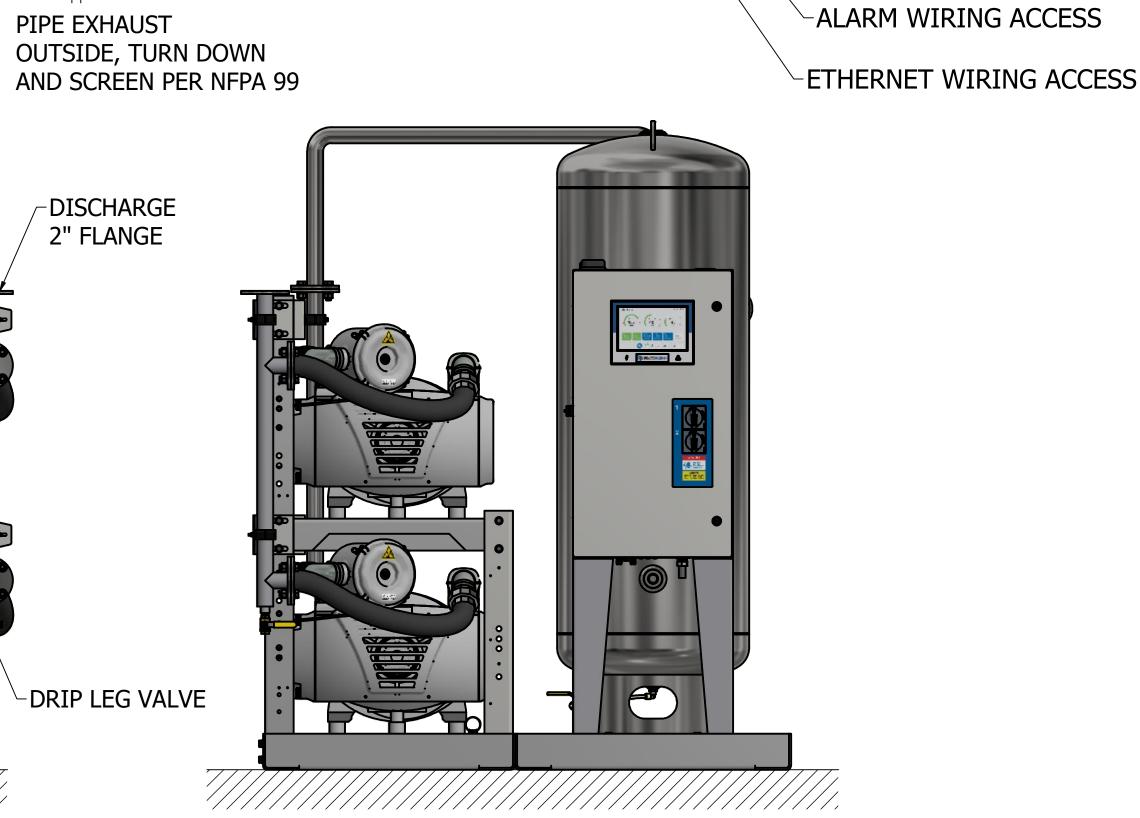

NO FOUNDATION OR INERTIA PAD IS REQUIRED (HOUSEKEEPING PAD IS OPTIONAL)

This drawing and the information contained thereon remain the property of BeaconMedaes and may not be used for other than the purpose for which it is loaned without the expressed written permission from BeaconMedaes Engineering. BEACONMEDÆS 4107 8556 12 INSTALLATION DIAGRAM DX 10HP HOP230844 DRY SPC V 120G TAE Sheet 1 of 1 DO NOT SCALE TH DOCUMENT

MAIN LINE VACUUM SWITCH

-FOR INTAKE PIPE

SIZING REFER TO STANDARD

PRESSURE DROP TABLES.

TOTAL PRESSURE DROP

SHOULD BE LESS THAN

VACUUM LEVEL OF 15" Hg.

2" Hg AT A SYSTEM

SINGLE POINT POWER

WIRING ACCESS

FROM HOSPITAL

FIELD PIPING INLET ====== FIELD PIPING OUTLET

EXHAUST PIPE SIZE TABLE (SEE NOTE 9)			
UNIT SIZE	PIPE LENGTH		
	0' - 300'	301' - 500'	
DPX 10 HP	3.00" NPT	4.00" NPT	
	•		

EFFECTIVE PIPE LENGTH TABLE FOR ELBOWS			
PIPE SIZE (IN)	3.00" NPT	4.00" NPT	
EFFECTIVE PIPE LENGTH (FT)	7.9	10.0	

Form F-007 Rev. 01 8