
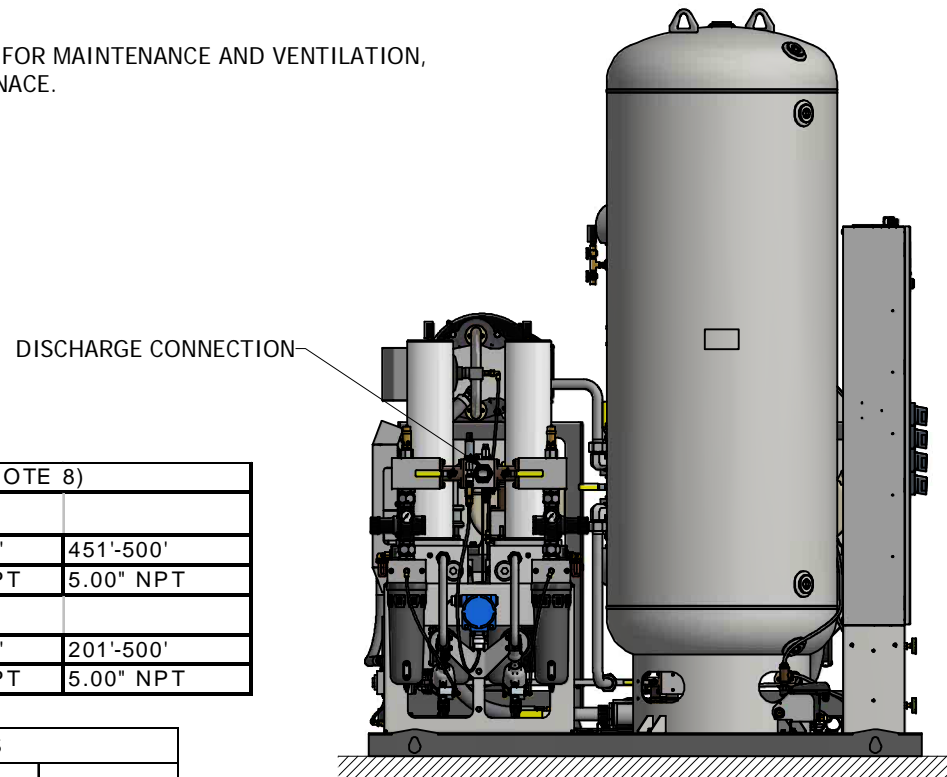


GENERAL NOTES:

1. ALL INTERCONNECTING PIPING, ACCESSORIES, AND PIPE FITTINGS SHALL BE SUPPLIED AND INSTALLED BY THE CONTRACTOR.
2. ALL PIPING SHALL BE HARD-DRAWN SEAMLESS TYPE "K" OR "L" COPPER TUBING CONFORMING TO ASTM DESIGNATION B-819.
3. ALL PIPING SHALL BE INSTALLED IN ACCORDANCE WITH NFPA 99, 2005 EDITION. PIPING LAYOUT IS DIAGRAMMATIC ONLY. FINAL ARRANGEMENT WILL BE SITE SPECIFIC. CONTRACTOR WILL BE RESPONSIBLE FOR FIELD VERIFICATION AND COORDINATING ACTUAL LOCATIONS WITH ALL OTHER TRADES.
4. CONTRACTOR SHALL VERIFY MOTOR VOLTAGE, PHASE AND AMP RATINGS BEFORE STARTING ELECTRICAL INSTALLATION AND MAKE CERTAIN THE VOLTAGE SUPPLIED BY THE HOSPITAL IS THE SAME.
5. THE ROOM REQUIRES DEDICATED MECHANICAL VENTILATION WITH AN ALLOWABLE TEMPERATURE VARIANCE OF 40° TO 105°F. (FOR HIGHER AMBIENTS, CONSULT FACTORY).
6. ELECTRICAL POWER SHOULD BE SUPPLIED FROM THE EQUIPMENT SYSTEM BRANCH OF THE ESSENTIAL ELECTRICAL SYSTEM (EMERGENCY POWER).
7. EQUIPMENT, INSTALLATION, AND TESTING OF THE MEDICAL AIR SYSTEM SHALL COMPLY WITH NFPA 99, 2005 EDITION, AND ALL STATE AND LOCAL CODES OR ORDINANCES.
8. WHEN DETERMINING THE TOTAL PIPE LENGTH, ADD ALL THE STRAIGHT LENGTHS OF PIPE TOGETHER IN ADDITION TO THE NUMBER OF ELBOWS TIMES THE EFFECTIVE PIPE LENGTH FOR THAT PIPE SIZE. (SEE EQUIVALENT PIPE LENGTH TABLE & EXAMPLE CALCULATION IN O & M MANUAL)
9. ALLOW 36 INCHES IN FRONT OF CONTROL PANEL FOR MAINTENANCE AND VENTILATION, ALL OTHER SIDES REQUIRE 24 INCHES OF CLEARANCE.

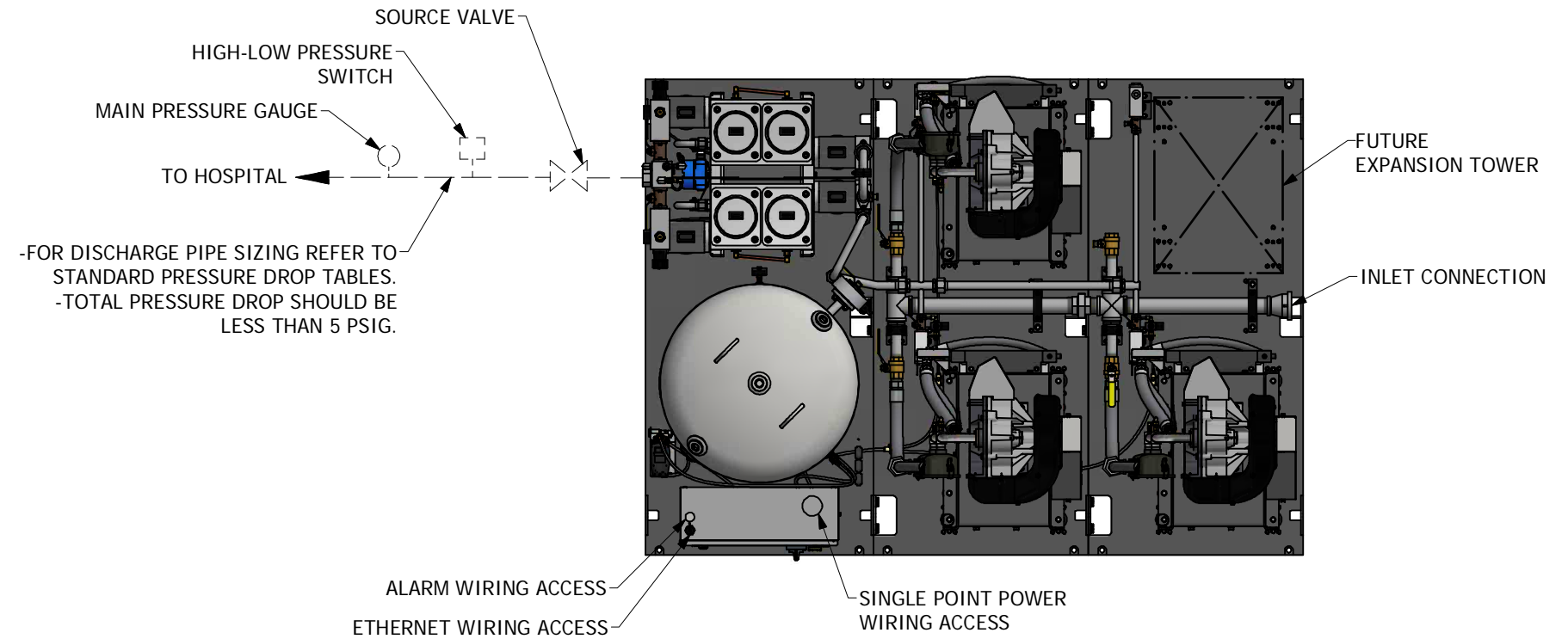


INTAKE PIPE SIZE TABLE (SEE NOTE 8)

| UNIT SIZE | PIPE LENGTH | | | |
|-----------|-------------|-----------|-----------|-----------|
| | 0-100' | 101'-250' | 251'-450' | 451'-500' |
| QPX 10 | 3.00" NPT | 3.50" NPT | 4.00" NPT | 5.00" NPT |
| UNIT SIZE | PIPE LENGTH | | | |
| | 0-50' | 51'-100' | 101'-200' | 201'-500' |
| QPX 15 | 3.00" NPT | 3.50" NPT | 4.00" NPT | 5.00" NPT |

EFFECTIVE PIPE LENGTH FOR ELBOWS

| PIPE SIZE (IN) | 3.00" NPT | 3.50" NPT | 4.00" NPT | 5.00" NPT |
|-----------------------|-----------|-----------|-----------|-----------|
| EFF. PIPE LENGTH (FT) | 7.9 | 9.4 | 10 | 11.9 |

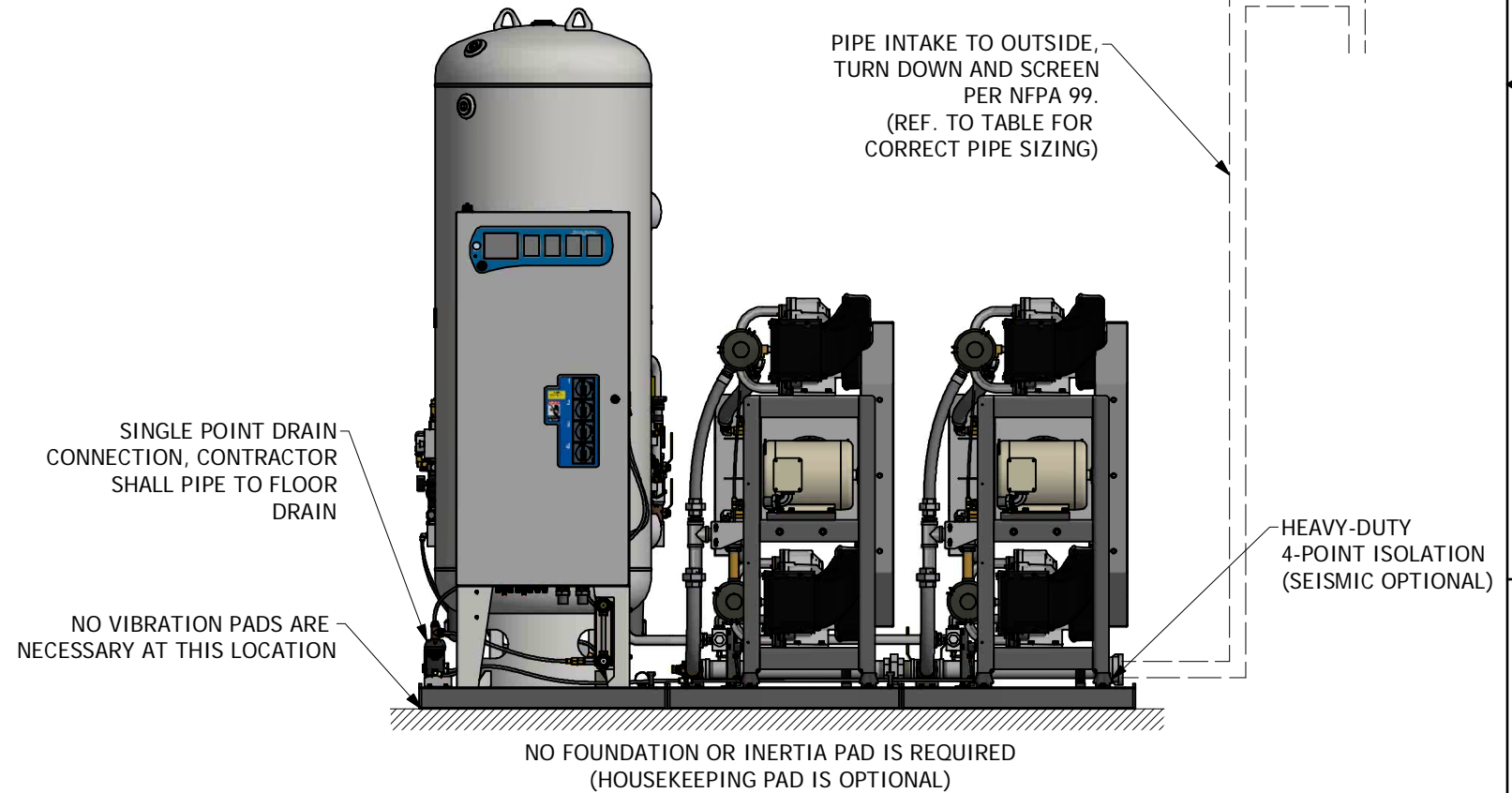

NOTE:
ALL ACCESSORIES, PIPE, AND FITTINGS BEYOND INLET AND DISCHARGE PORTS ARE SUPPLIED AND INSTALLED BY OTHERS.

(15HP WITH 240 GALLON RECEIVER SHOWN)

| | | | |
|--|------------------|--|----------------------------|
| Revised: MDB | Date: 05/28/2019 | Drawn: jeb | Date: 11/19/2009 |
| | | This drawing and the information contained therein remain the property of BeaconMedes and may not be used for other than the purpose for which it is loaned without the expressed written permission from BeaconMedes Engineering. | |
| Description: INSTALLATION DIAGRAM 10-15HP TX-EXP-OX SCROLL SPC | | Part Number: 4107 8511 95 | Scale: 1:12 |
| DN: BM190342 | Rev: 02 | Sheet 1 of 1 | DO NOT SCALE THIS DOCUMENT |