



Type the default password (121212) and touch Enter. Choose continue or change password.

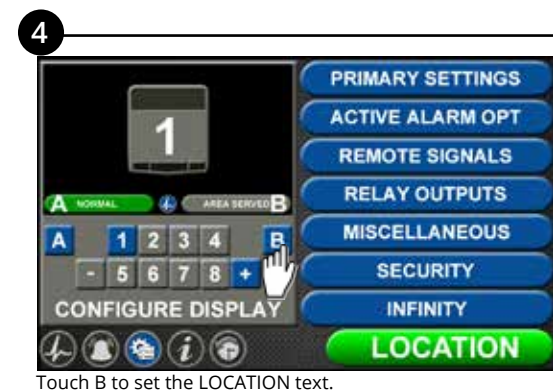

Touch B to set the LOCATION text.

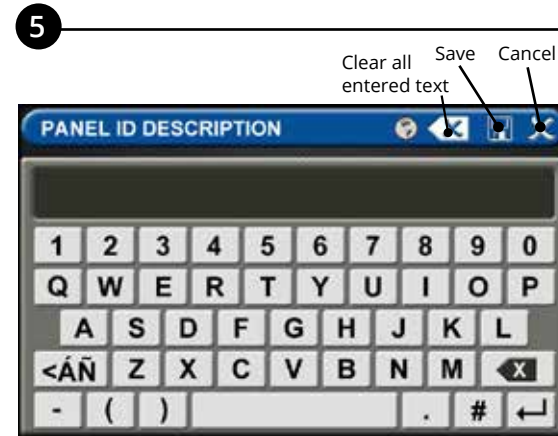

Enter area served and touch save.



On the configuration screen, touch the blue 1 to configure the area monitored for the first gas badge.



Touch a blank location label to enter new text, or choose an existing location label if the location has already been set up. Touch save. Repeat steps 6 and 7 for additional gases.

Product Identification

Each alarm is identified by a Model number, Part number, and Lot Code.

Installation procedures vary depending on the alarm configuration.

The product ID label is located on the inside of the alarm back box (Figure 1).

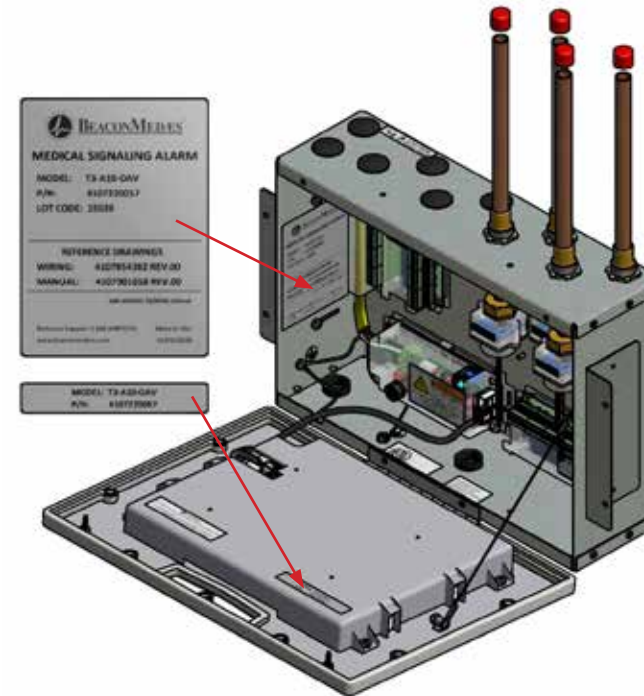


Figure 1: Product Identification Labels

NOTE:

If gas sensor wires are landed on the terminal blocks in the same order as defined by the model number of the alarm panel, then initial setup will be easier and gas sensor channels will not need to be reconfigured." Example T3-A10-OXACVNW means "O" Oxygen is defaulted to D01, "X" Nitrous Oxide is defaulted to D02, "A" Medical Air is defaulted to D03, "C" Carbon Dioxide is defaulted to D04, "V" Medical Vacuum is defaulted to D05, "N" Nitrogen is defaulted to D06, and "W" WAGD is defaulted to D07. See Table.

Gas Type (Area and Combination Only)	
Allowable Value	Description
O	Oxygen
X	Nitrous Oxide
A	Medical Air
V	Medical Vacuum
W	WAGD
N	Nitrogen
C	Carbon Dioxide (CO2)
D	Oxygen 100 psig
F	Medical Air 100 psig
G	Carbon Dioxide 100 psig
9	Instrument Air
1	CO2-O2 (CO2 over 7%)
2	O2-CO2 (CO2 not over 7%)
3	HE-O2 (HE over 80%)
4	O2-HE (HE not over 80%)
7	Laboratory Air
8	Laboratory Vacuum
H	Helium
J	Argon
S	Surgical Air
B	AGSS
E	N2O-O2
M	Mixed Gas

Quick Setup Guide: TotalAlert Infinity™/Mega3 Area Alarm




Part number 4107 9016 93
 Revision 04
 February 16, 2022

This guide is intended for Quick Setup of the TotalAlert Infinity™/Mega3 area alarm. For more detailed instructions, see the Operation and Maintenance Manual.

Alarm documentation, including the operation manual, is available at www.beacontai.com.



Wiring Diagram for NFPA TotalAlert/Mega3 area alarm

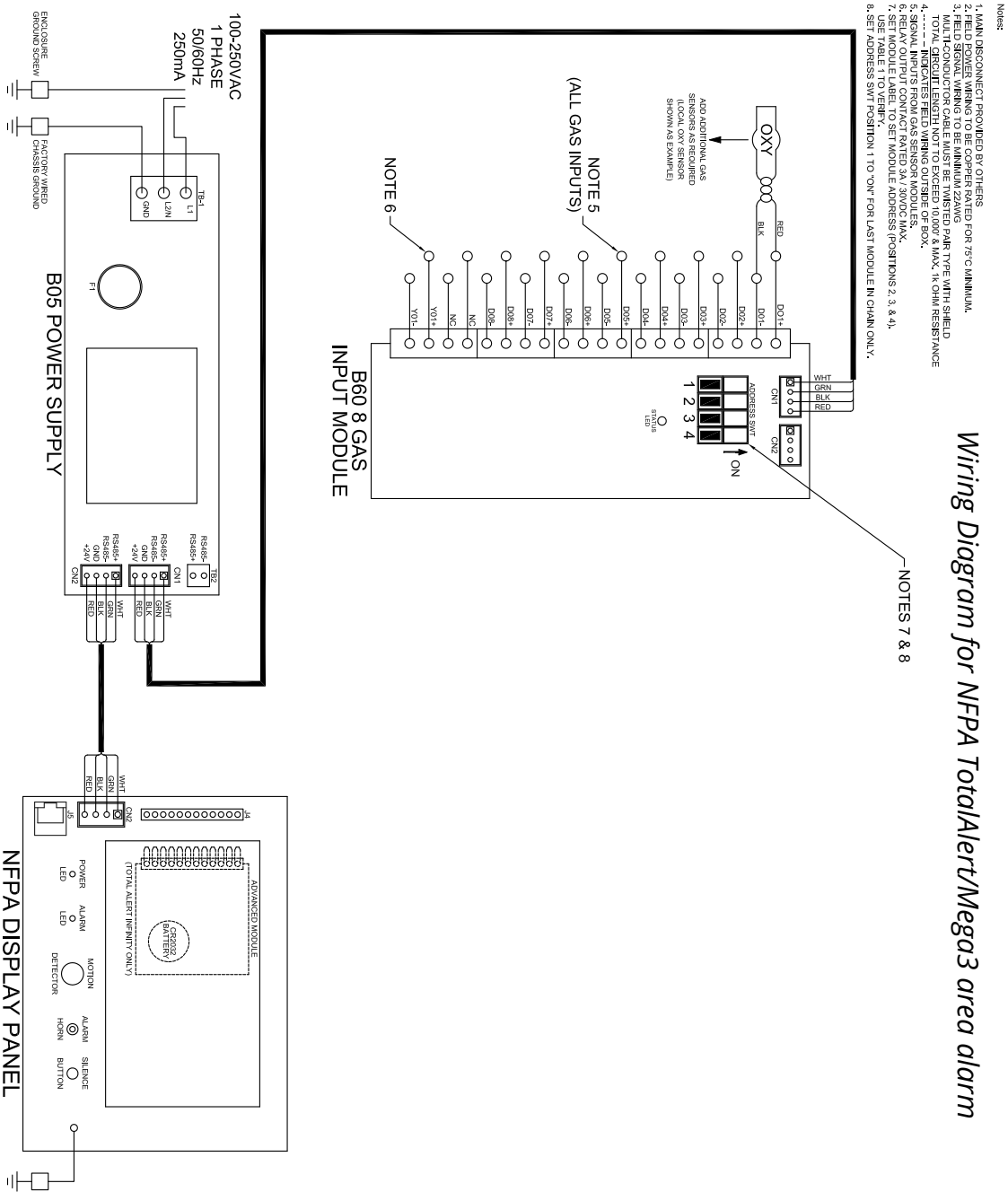

WARNING:
RISK OF ELECTRIC SHOCK
 Disconnect power at the circuit breaker before removing power supply shield.